
**OPERATOR'S MANUAL - ENGLISH
MANUAL DE INSTRUCCIONES -
DOMESTIC SPANISH**


**INVACARE 3
INVACARE 5
INVACARE 6**

**OXYGEN CONCENTRATORS STANDARD,
WITH /SENSO₂[®],
TELEMETRY AND HOMEFILL™ II
CONCENTRADORES DE OXIGENO
STANDARD, *CON /SENSO₂*[®],
TELEMETRIA Y
HOMEFILL™ II**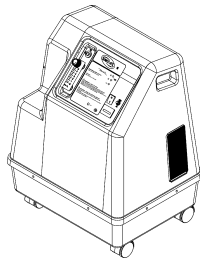

**Invacare 5 *with/SensO₂*[®]
Model IRC5LXO2**

**Invacare 5 *con/SensO₂*[®]
Modelo IRC5LXO2**

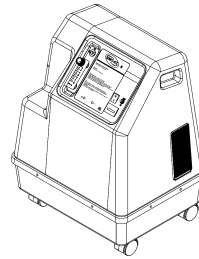**Invacare 5
Model IRC5LX**

**Invacare 5
Modelo IRC5LX**

DEALER: THIS MANUAL MUST BE GIVEN
TO THE END USER.

USER: BEFORE USING THIS
CONCENTRATOR,
READ THIS MANUAL AND SAVE FOR
FUTURE REFERENCE.

Yes, you can:

SPECIAL NOTES

SPECIAL NOTES

WARNING/CAUTION notices used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

NOTICE
THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

SAVE THESE INSTRUCTIONS

TABLE OF CONTENTS

TABLE OF CONTENTS

SPECIAL NOTES	2
SAFETY SUMMARY	4
FEATURES	8
SPECIFICATIONS	9
HANDLING	12
Unpacking	12
Inspection	12
Storage	12
INTRODUCTION	13
OPERATING INSTRUCTIONS	13
Select a Location	13
Recommended End-User Guidelines for Optimum Performance	14
Plug in Power Cord	14
Connect Humidifier (if so prescribed)	14
Oxygen Tubing	15
Power Switch	16
Flowrate	16
SensO ₂ [®] Oxygen Purity Indicator- If your Unit Features the O ₂ Sensor Feature	16
Initial Startup of the Concentrator	17
Explanation of Indicator Lights	17
MAINTENANCE	18
Cleaning the Cabinet Filters	18
Cabinet	19
Humidifier	19
TROUBLESHOOTING GUIDE	20
PREVENTIVE MAINTENANCE RECORD	21
WARRANTY	22
SPANISH OWNER'S MANUAL (MANUAL DEL USARIO)	24

SAFETY SUMMARY

SAFETY SUMMARY

CAUTION

“Caution: Federal law restricts this device to sale by or on order of a physician, or any other practitioner licensed by the law of the State in which he practices to use or order the use of this device.”

Risk of electric shock. **DO NOT** disassemble. Refer servicing to qualified service personnel.

READ THE FOLLOWING INFORMATION BEFORE OPERATING THIS PRODUCT.

WARNING

The use of oxygen therapy requires that special care be taken to reduce the risk of fire. Any materials that will burn in air, and some that will not, are easily ignited and burn rapidly in high concentrations of oxygen. For safety concerns, it is necessary that all sources of ignition be kept away from the product and preferably out of the room in which it is being used. **NO SMOKING** signs should be prominently displayed.

A spontaneous and violent ignition may occur if oil, grease or greasy substances come in contact with oxygen under pressure. These substances **MUST** be kept away from the oxygen concentrator, tubing and connections, and all other oxygen equipment. **DO NOT** use any lubricants unless recommended by Invacare.

For optimum performance, Invacare recommends that each concentrator be on and running for a minimum of 30 minutes at a time. Shorter periods of operation may be harmful for maximum product life. If it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water, call **Qualified Service Personnel** for examination and repair.

Keep the cord away from **HEATED** or **HOT** surfaces.

NEVER drop or insert any object into any opening.

SAFETY SUMMARY

WARNING (CONTINUED)

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air opening may be blocked. Keep the openings free from lint, hair and the like. Fill humidifier with water to the level shown by the manufacturer. **DO NOT** overfill.

Invacare recommends that Crush-Proof oxygen tubing (supplied by Invacare) be used with this product and **NOT** exceed 50 ft. (15.2m) in length.

MAINTENANCE

The Invacare Oxygen Concentrator was specifically designed to minimize routine preventative maintenance at intervals of once per year. Only professionals of the healthcare field or persons fully conversant with this process such as authorized or factory trained personnel should perform preventative maintenance or performance adjustments on the oxygen concentrator.

RADIO FREQUENCY INTERFERENCE

Most electronic equipment is influenced by Radio Frequency Interference (RFI). **CAUTION** should be exercised with regard to the use of portable communications equipment in the area around such equipment.

TO REDUCE THE RISK OF ELECTROCUTION.

WARNING

ALWAYS unplug this product **IMMEDIATELY** after using. Avoid using while bathing. If continuous usage is required by the physicians prescription:

The concentrator must be located in another room at least seven (7) feet from the bath.

DO NOT come in contact with the concentrator while wet.

DO NOT place or store product where it can drop into water or other liquid.

DO NOT reach for product that has fallen into water. **UN-PLUG IMMEDIATELY.**

SAFETY SUMMARY

TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS.

WARNING

This device is to be used only in accordance with the prescription of a physician and this Owners Manual. If at any time the patient or attendant conclude that the patient is receiving an insufficient amount of oxygen, the supplier and/or physician should be contacted immediately. No adjustments should be made to the flowrate unless prescribed by a physician.

A product should **NEVER** be left unattended when plugged in.

Close supervision is necessary when this product is used by, on or near children or bed confined individuals.

Use this product for only intended use as described in this manual.

DO NOT use parts, accessories or adapters other than those authorized by Invacare or warranty will be voided.

DO NOT connect the concentrator in parallel or series with other oxygen concentrators or oxygen therapy devices, or warranty will be void.

Use of some administration accessories or certain humidifiers not specified for use with oxygen concentrator may impair the performance.

In certain circumstances oxygen therapy can be hazardous. Invacare recommends that you seek medical advice before using this machine.

This equipment is to be used as an oxygen supplement and is not considered life supporting or life sustaining.

Avoid creation of any spark near medical oxygen equipment.

This includes sparks from static electricity created by any type of friction.

SAFETY SUMMARY

GROUNDING INSTRUCTIONS


- A. This product should be grounded. In the event of an electrical short circuit, grounding reduces the risk of electric shock by providing an escape wire for the electrical current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

DANGER

Improper use of the grounding plug can result in a risk of electrical shock.

If repair or replacement of the cord or plug is necessary, do not connect the grounding wire to either flat blade terminal. The wire with insulation having an outer surface that is green with or without yellow stripes is the grounding wire. Check with a qualified electrician or serviceman if the grounding instructions are not completely understood, or if doubt exists as to whether the product is properly grounded.


- B. This product is for use on a normal 120 V circuit and has a grounding plug that looks like the plug illustrated in sketch A shown at the bottom of the page. A temporary adapter, which looks like the adaptor illustrated in sketches B and C, may be used to connect this plug to a 2-pole outlet as shown in sketch C if a properly grounded outlet is not available. The temporary adaptor should be used only until a properly grounded outlet (sketch D) can be installed by a qualified electrician. The green colored rigid ear, lug and the like extending from the adapter must be connected to a permanent ground such as properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by the screw.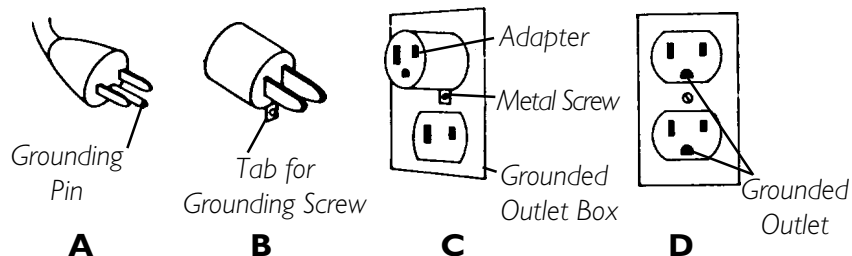


FEATURES

FEATURES

FRONT VIEW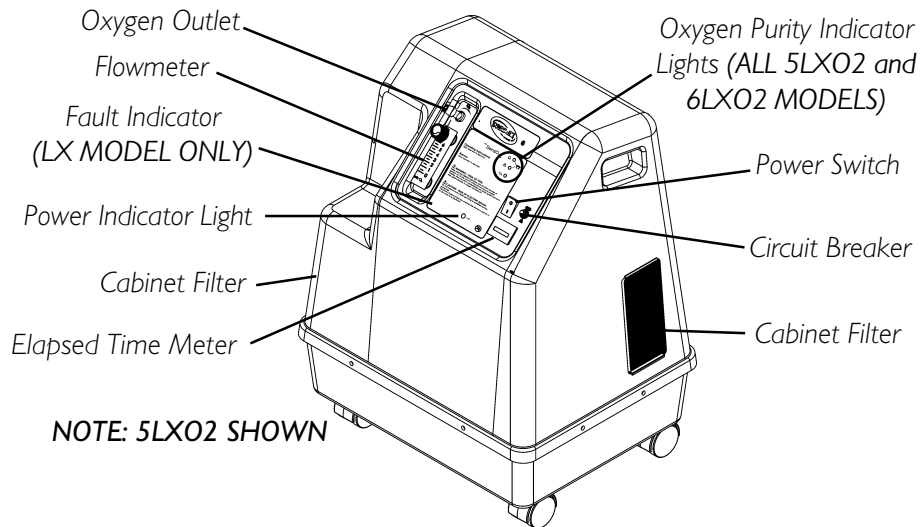

REAR VIEW

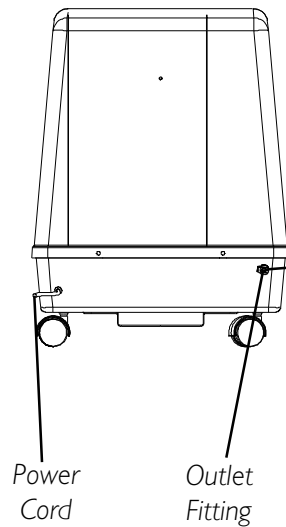ACCESSORIES (NOT SHOWN):

Remote Humidifier/Flowmeter
Accessory - RF5

PreciseR_x[™] Pediatric Humidifier/
Flowmeter Accessory - PF16

Venture[®] Homefill[™] II home oxygen
compressor - IOH200

NOTE: This outlet fitting is to be used only for filling oxygen cylinders with the optional Venture HomeFill II home oxygen compressor. The outlet fitting does not affect concentrator performance. Refer to the HomeFill II owner's manual, part number 1100873, for connection and operating instructions. When not in use, the plug provided with the concentrator should be inserted into the outlet fitting. For more information about the HomeFill II, contact your Invacare dealer.

SPECIFICATIONS

SPECIFICATIONS


Electrical Requirements:	120 VAC \pm 10% (132 VAC/108 VAC), 60 Hz
Rated Current Input:	4.0 A
Sound Level:	52 dB Average (3LX) 53 dB Average (ALL 5LX/5LXO2/ 6LXO2)
Altitude:	Up to 6,000 ft. (1828 meters) above sea level without degradation of performance. From 6,000 ft. (1828 meters) to 13,129 ft. (4000 meters) below 90% efficiency.
*Oxygen Output Concentration Levels:	ALL 6LXO2 / 5LXO2 / 3LX MODELS 95.6% to 87% at all flowrates 95.6% to 93% O2 @ 2L/min. 95.6% to 87% O2 @ 3L/min. (3LX - Maximum recommended flow) 95.6% to 87% O2 @ 5L/min. (5LX - Maximum recommended flow) 95.6% to 87% O2 @ 6L/min. (6LX - Maximum recommended flow) * Stated performance maximum achieved after 30 min. of run time.
Maximum Outlet Pressure:	(ALL 6LXO2) 5 \pm 0.5 psi. (34.5 \pm 3.45 kPa.) (ALL 5LX/5LXO2) 5 \pm 0.5 psi. (34.5 \pm 3.45 kPa.) (3LX) 5 \pm 0.5 psi. (34.5 \pm 3.45 kPa.)

SPECIFICATIONS

SPECIFICATIONS (CONTINUED)

SensO₂[®] Alarm Thresholds:

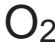INDICATORS

Label Symbol	O ₂ Purity	Internal Switch (If Present) *Set at 73%	Internal Switch (If Present) Set at 85%
	Over 85% (± 2%)	GREEN Indicator Light	GREEN Indicator Light
	73% (± 3%) to 85% (±2%)	YELLOW Indicator Light	RED Indicator Light - Continuous Audible Alarm Sieve-GARD™ Compressor Shutdown
	Below 73% (± 3%)	RED Indicator Light - Continuous Audible Alarm Sieve-GARD™ Compressor Shutdown	N/A

* Factory Preset at 73%

Flow Range:

1/2 - 6 L/min. (**ALL 6LXO2**)

1/2 - 5 L/min. (**ALL 5LX/5LXO2**)

1/2 - 3 L/min. (**3LX**)

For flowrates less than 1 L/min., we recommend the use of the Invacare Pediatric Flowmeter IRCPF16.

Average Power Consumption:

400W (**ALL 3LX/5LX/5LXO2**)

460W (**ALL 6LXO2**)

Pressure Relief Mechanism Operational at:

35 psi ± 3.5 psi (24.1 kPa ± 24.1 kPa)

Change in maximum recommended flow when back pressure of 7kPa is applied:

0.7 L/m

Filters:

Cabinet, Outlet HEPA Filter and Compressor Inlet.

SPECIFICATIONS

SPECIFICATIONS (CONTINUED)

Safety System:	Current overload or line surge shutdown. High temperature compressor shutdown. High Pressure Alarm w/compressor shutdown. Low Pressure Alarm w/compressor shutdown. Battery Free™ Power Loss Alarm. SensO ₂ ® Oxygen System (ALL 5LXO2 and 6LXO2 Models).
Width:	18-inches (45.7 cm).
Height:	23.75-inches (60.3 cm).
Depth:	14-inches (35.6 cm).
Weight:	54 lbs. (24.5 kg) (5LX and ALL LXO2 Models) 53 lbs. (23.9 kg) (3LX)
Shipping Weight:	59 lbs. (27 kg) (5LX and ALL LXO2 Models) 58 lbs. (26.1 kg) (3LX)
Operating Ambient Temperature:	50° F - 95° F (10° C - 35° C).
Exhaust:	Less than Ambient + 35° F (+16°C).
Oxygen Output:	Less than Ambient + 4° F (+2° C).
Cabinet:	Impact Resistant flame-retardant plastic cabinet.
Regulatory Listing:	ETL certified complying with UL 1431 (U.S. Models). CSA certified to C22.2 No. 125-M1984 (Canadian Models).
Electrical:	No extension cords.
Placement:	No closer than 3-inches from any wall.
Tubing:	Crush-Proof Tubing - 7 ft. (2.1m) cannula with a maximum 50 ft. (15.2m) extension (DO NOT PINCH).
Relative Humidity:	20 to 60%
Time of Operation:	Up to 24 hours per day.
Recommended Storage and Shipping Temperature:	-20°F to 150°F (-29°C to 65°C)

SPECIFICATIONS

HANDLING

The concentrator should always be kept in the upright position to prevent cabinet damage while being transported. The shipping container has been designed to assure maximum protection of the concentrator. If the concentrator is to be reshipped by common carrier, additional cartons are available from Invacare.

UNPACKING

1. Check for any obvious damage to the carton or its contents. If damage is evident, notify the Carrier, or your local dealer.
2. Remove all loose packing from the carton.
3. Carefully remove all the components from the carton.

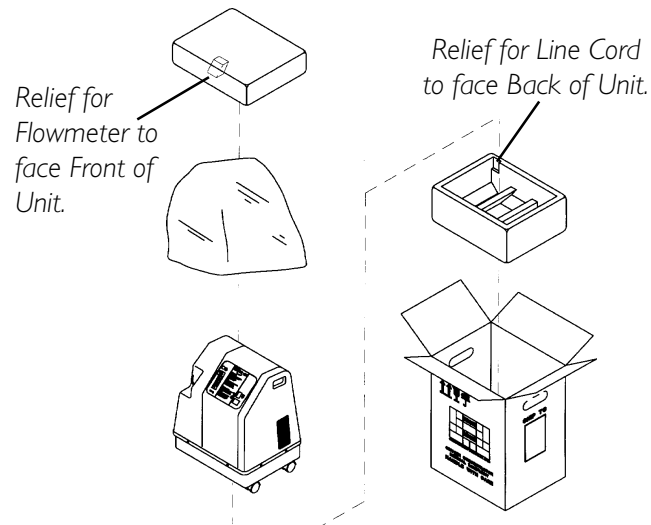
FIGURE I - PACKAGING

NOTE: Unless the Oxygen Concentrator is to be used immediately, retain containers and packing materials for storage until use of the concentrator is required.

INSPECTION

1. Examine exterior of the Oxygen Concentrator for nicks, dents, scratches or other damages. Inspect all components.

STORAGE

1. Store the repackaged Oxygen Concentrator in a dry area.
2. DO NOT place other objects on top of the repackaged concentrator.

HANDLING (CONTINUED)

INTRODUCTION

Your Oxygen Concentrator is intended for use in the home. It is an electronically operated device that separates oxygen from room air. It provides high concentration of oxygen directly to you through a nasal cannula or mask. Clinical studies have documented that oxygen concentrators are therapeutically equivalent to other types of oxygen delivery systems.

Your Dealer will show you how to use your oxygen concentrator. He/She should be contacted with any questions or problems regarding your Oxygen Concentrator. This Owners Manual will tell you about your concentrator and will serve as a reference as you use your concentrator.

OPERATING INSTRUCTIONS

SELECT A LOCATION

You may select a room in your house where using your oxygen concentrator would be most convenient. Your concentrator can be easily rolled from room to room on its casters.

Your Oxygen Concentrator will perform best when operated under the following conditions. Usage in environments other than those described may result in the need for increased equipment maintenance. The air intake of the unit should be located in a well ventilated area to avoid airborne pollutants and/or fumes.

Recommended End-User Guidelines for Optimum Performance:

Temperature:	50°F - 95°F (10°C - 35°C)
Electrical:	No extension cords.
Placement:	No closer than 3-inches from the wall.
Tubing and Cannula:	57 feet maximum (17.3m) of Crush-Proof Tubing (do NOT pinch).
Environment:	Smoke and soot-free. No confined spaces (Example: no closets).
Relative Humidity:	20 to 60 %
Time of Operation:	Up to 24 hours per day.
Flow Rate - 3LX:	From 1/2 L/min. to 3 L/min.
All 5LX/5LXO2 Models:	From 1/2 L/min. to 5 L/min.

HANDLING

HANDLING (CONTINUED)

All 6LXO2 Models: From 1/2 L/min. to 6 L/min.
For flow rates less than 1 L/min., use the Pediatric Flowmeter.

Minimum Operating Time: 20 Minutes

NOTE: Ensure that your concentrator is at least three (3) inches away from walls, draperies or furniture to assure sufficient air flow. Avoid deep pile carpets and heaters, radiators or hot air registers.

PLUG IN POWER CORD

1. Plug in power cord to a properly grounded outlet.

CONNECT HUMIDIFIER (IF SO PRESCRIBED)

1. Fill humidifier with distilled water to the level indicated by the manufacturer.

WARNING

DO NOT overfill humidifier.

Replace the humidifier cap and securely tighten.

DO NOT reverse the oxygen input and output connections.

Water from the humidifier bottle **WILL** travel through the cannula back to the patient.

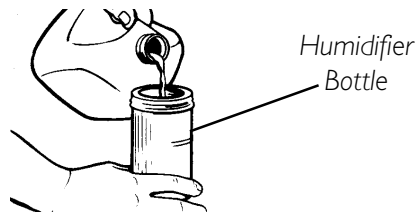
FIGURE 1 - FILLING THE HUMIDIFIER

2. Place the humidifier in the humidifier compartment.

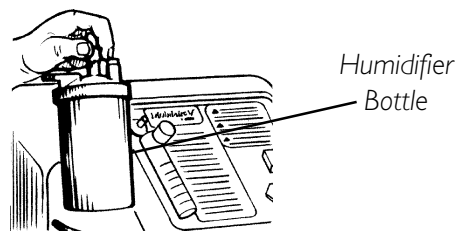
FIGURE 2 - HUMIDIFIER COMPARTMENT

OPERATION / MAINTENANCE

3. Attach the humidifier adapter by turning the wing nut on the humidifier bottle counterclockwise until it is securely fastened.

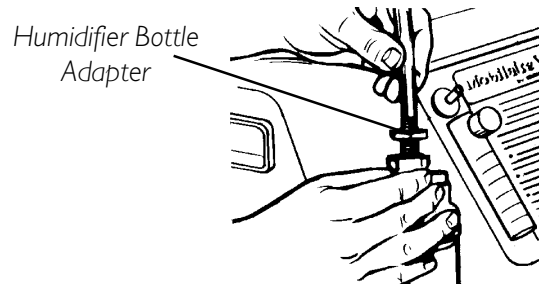
FIGURE 3 - ATTACHING THE HUMIDIFIER ADAPTER

4. If using the Remote Humidifier/Flowmeter Accessory, refer to the Operating Instructions which accompany the accessory.
5. After assembly, ensure that oxygen is flowing through the cannula.

OXYGEN TUBING

1. Attach oxygen tubing from the humidifier bottle to oxygen outlet connector on the oxygen concentrator.

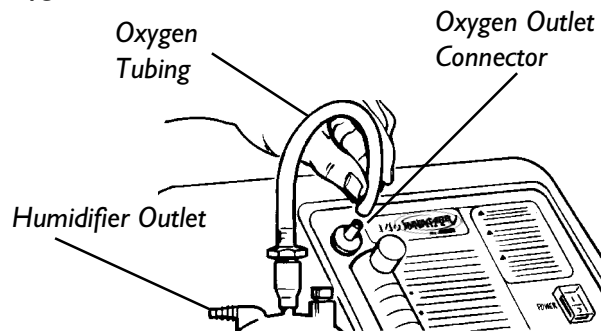
FIGURE 4 - ATTACHING THE OXYGEN TUBING

2. Attach the patient supply tubing to the humidifier bottle.

WARNING

For optimum performance, Invacare recommends that concentrator be operated a minimum of 20 minutes during any use. Shorter periods of operation may adversely affect the life of the product.

OPERATION / MAINTENANCE

POWER SWITCH

1. Press power switch to the **ON** position. The power light (I/O), located on the control panel, will illuminate a **GREEN** light. The alarm will "beep" three (3) times indicating that the system is functioning properly.

FLOWRATE

1. Turn the flowrate knob to the setting prescribed by your physician or therapist.

WARNING

DO NOT change the L/min. setting on the flowmeter unless a change has been prescribed by your physician or therapist.

NOTE: To properly read the flowmeter, locate the prescribed flowrate line on the flowmeter. Next, turn the flow knob to until the ball rises to the line. Now, center the ball on the L/min. line prescribed.

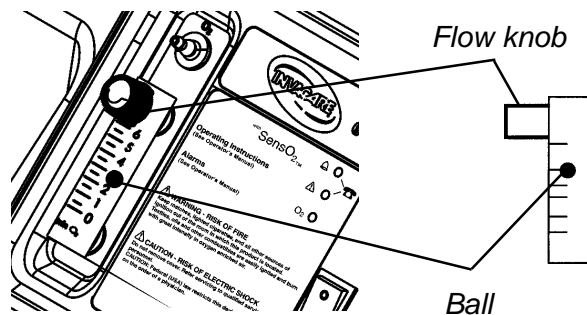
FIGURE 5 - ADJUSTING THE FLOWRATE

SENSO₂[®] OXYGEN PURITY INDICATOR-IF YOUR UNIT FEATURES THE O₂ SENSOR FEATURE

This feature monitors the purity of oxygen generated by the oxygen concentrator. If purity falls below factory preset standards, indicator lights on the control panel will illuminate.

OPERATION / MAINTENANCE

INITIAL STARTUP OF THE CONCENTRATOR

NOTE: Concentrator may be used during initial warm-up time (approximately 30 min.) while waiting for the GREEN light to illuminate.

When the unit is initially turned on, the **GREEN** power LED (I/O) will illuminate. Within 5 to 30 minutes after initial startup, a **GREEN** light will illuminate in the **Oxygen Purity** section of the **Control Panel (FIGURE 6)** if concentration is within specification. Any other SensO₂[®] indicator lights illuminating will signal a condition that requires a call to the supplier.

OPERATION / MAINTENANCE

EXPLANATION OF INDICATOR LIGHTS:

GREEN LIGHT (O₂) - Normal Operation

YELLOW LIGHT (⚠) - You may continue to use the concentrator unless instructed otherwise by your supplier. Call Supplier **IMMEDIATELY**. Be certain that **BACKUP OXYGEN** is nearby.

RED LIGHT (🔔) - Total unit shutdown. Switch **IMMEDIATELY** to **BACKUP OXYGEN** supply. Call Supplier **IMMEDIATELY**.

GREEN LIGHT - with **YELLOW LIGHT FLASHING** - Oxygen sensor malfunctioning; you may continue to use the concentrator. Call Supplier **IMMEDIATELY**.

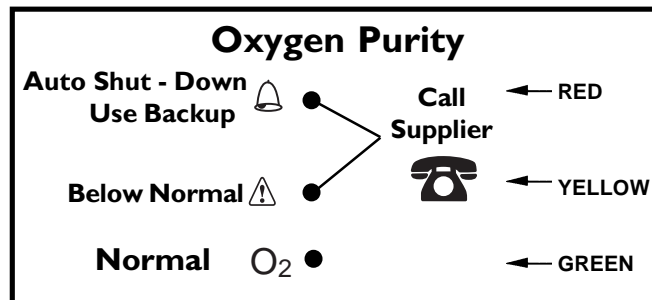
FIGURE 6 - O₂ DISPLAY CONTROL PANEL

NOTE: LX models built AFTER 12/2000 include a red "fault" indicator (🔔) on the control panel. Switch immediately to a back-up oxygen supply and call supplier when illuminated.

OPERATION / MAINTENANCE

MAINTENANCE CLEANING THE CABINET FILTERS

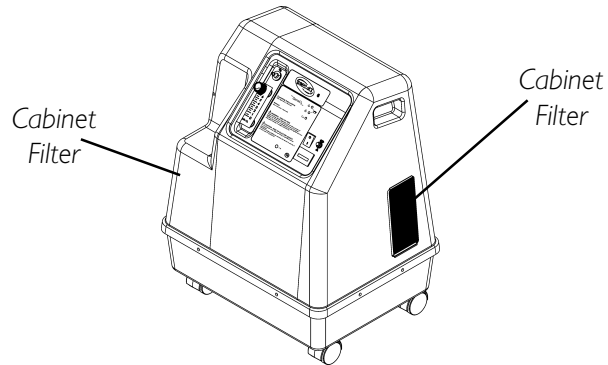
FIGURE 7 - CABINET FILTER LOCATION

NOTE: All concentrator Models have two (2) cabinet filters located on each side of the cabinet (FIGURE 7).

1. Remove each filter and **CLEAN AT LEAST ONCE A WEEK** depending on environmental conditions (FIGURE 8).
2. Clean the cabinet filters with a vacuum cleaner or wash in warm soapy water and rinse thoroughly (FIGURE 8).
3. Dry the filters thoroughly before reinstallation.

CAUTION

DO NOT operate the concentrator without the filters installed.

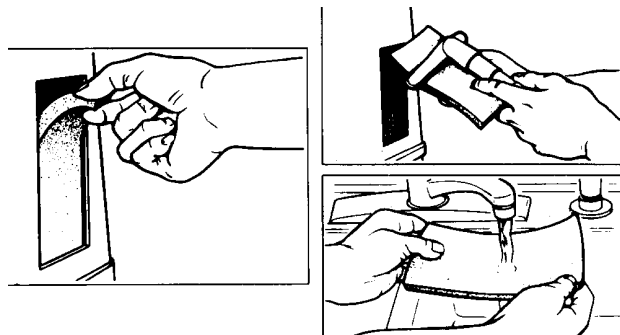
FIGURE 8 - CLEANING THE CABINET FILTERS

OPERATION / MAINTENANCE

CABINET

WARNING

UNPLUG the concentrator when cleaning. To avoid electrical shock, **DO NOT** remove cabinet.

1. Clean the cabinet with a mild household cleaner and non-abrasive cloth or sponge.

HUMIDIFIER

NOTE: To clean the oxygen humidifier, follow the instructions provided by the manufacturer. If none are provided, follow these steps:

1. Clean the humidifier every day.
2. Wash it in soapy water and rinse it with a solution of 10 parts water and 1 part vinegar.
3. Rinse again with hot water and refill with distilled water to the level shown on the humidifier.

WARNING

DO NOT overfill Humidifier Bottle.

TROUBLESHOOTING

TROUBLESHOOTING GUIDE

<p>Alarm: SHORT BEEPS, LONG PAUSE</p> <p>Concentrator NOT operating, power switch ON. Beep..... Beep.....</p>	<p>Main Power Loss:</p> <ol style="list-style-type: none"> 1. Power cord NOT plugged In. 2. No power at outlet. 3. Tripped circuit breaker. 	<ol style="list-style-type: none"> 1. Insert plug into outlet. 2. Inspect house circuit breakers or fuses. If problem recurs, use a different outlet. 3. Call Dealer for Service.
<p>Alarm: CONTINUOUS Power Switch ON,</p> <p>Concentrator NOT OPERATING. Beeeeeeep....</p>	<p>System Failure:</p> <ol style="list-style-type: none"> 1. Unit overheating due to blocked air intake. 2. Insufficient power at outlet. 3. Internal repairs required. 	<ol style="list-style-type: none"> 1a. Remove and clean cabinet filters. 1b. Move oxygen concentrator at least 3-inches away from walls, draperies or furniture. 2. Don't use extension cords. Move to another electrical outlet or circuit. 3. Call Dealer for Service.
<p>YELLOW or RED LIGHT Illuminated.</p>	<ol style="list-style-type: none"> 1. Low oxygen purity. 2a. Kinked or blocked tubing, cannula or humidifier. 2b. Flowmeter set at 1/2 L/min.or less. 3. Unit overheating due to blocked air intake. 4. Insufficient power at outlet. 5. Internal repairs required. 	<ol style="list-style-type: none"> 1. Clean or Replace filters. 2a. Inspect for kinks or blockages. Correct, clean or replace item. Once corrected, turn power OFF for 60 seconds and then turn powerback ON. 3a. Remove and clean cabinet filters. 3b. Move concentrator at least 3-inches from walls, draperies or furniture. 4. Don't use extension cords. Move to another electrical outlet or circuit. 5. Call Dealer for Service.
<p>GREEN LIGHT with YELLOW Light Flashing.</p>	<ol style="list-style-type: none"> 1. Internal repairs required. 	<ol style="list-style-type: none"> 1. Call Dealer for Service.

**PREVENTIVE MAINTENANCE RECORD
INVACARE OXYGEN CONCENTRATOR**

ON EACH INSPECTION MODEL NO. IRC _____ SERIAL NO. _____

Record Date of Service																				
Record Elapsed Hours On Hour Meter																				
Clean Cabinet Filter(s)																				
Check Prescribed L/min. Flow Rate																				
EVERY 90 DAYS																				
Check Oxygen Concentration (UNITS WITHOUT SensO ₂ ®)																				
ANNUALLY OR DURING PREVENTIVE MAINTENANCE SCHEDULE																				
Clean/Replace Cabinet Filter(s)																				
Check Outlet HEPA Filter																				
Check Compressor Inlet Filter																				
Check Oxygen Concentration (SensO ₂ ® MODELS)																				
Check Power Loss Alarm																				
AS REQUIRED																				
Clean Heat Exchanger																				
Replace Exhaust Muffler																				
Rebuild Top End of Compressor																				
EVERY 20,000 HOURS																				
Replace In-line Filter - (ALL MODELS EXCEPT 3LX, 5LX, 5LXO2, 5LXO2T, 6LXO2, 6LXO2T)																				

PREVENTIVE MAINTENANCE RECORD

WARRANTY

LIMITED WARRANTY

NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to original purchaser of our products. This warranty gives you specific legal rights and you may also have other legal rights which may vary from state to state.

Invacare Corporation warrants its Oxygen Concentrators to be free from defects in materials and workmanship from date of purchase:

ALL Careguard 3LX Models AFTER 7/1/98 for a period of three (3) years

ALL Invacare 5LX, 5LXO2, and 6LXO2 Oxygen Concentrator Models for a period of five (5) years

The 4-way valve body carries a lifetime warranty from date of purchase. **DO NOT OPEN OR ATTEMPT TO SERVICE VALVE BODY.** This will void any and all warranty applicable to the valve body. If within such warranty period any such product shall be proven to Invacare Corporation's satisfaction to be defective, such product shall be repaired or replaced, at Invacare Corporation's option. This warranty only applies to the labor for repairs performed by the Invacare Service Department or authorized Invacare dealers. It does not apply to the labor performed by the purchaser or user. This warranty does not include normal wear and tear or shipping charges incurred in replacement part installation or repair of any such product. Invacare Corporation's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. Routine maintenance items, such as filters, are excluded from this warranty.

For warranty service, please contact Invacare Corporation's service department at the toll free number on the back cover.

Upon receiving notice of an alleged defect in a product, Invacare Corporation will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return the entire unit or remove, at purchaser's cost, the defective component part(s) identified, pack the component part(s) in a manner to avoid shipping damage and to ship the component part(s) to either Invacare Corporation's plant or service center as specified by Invacare Corporation in advance. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. Do not return products to our factory without prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

WARRANTY

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, SOOT OR SMOKE-FILLED ENVIRONMENTS, COMMERCIAL USE, OR OTHER THAN NORMAL APPLICATION, USE OR SERVICE, OR TO PRODUCTS MODIFIED WITHOUT INVACARE CORPORATION'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE CORPORATION OR TO PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE CORPORATION'S CONTROL.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN, AND THE REMEDY FOR VIOLATION OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE CORPORATION SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Some states do not allow the exclusion or limitation of incidental or consequential damage, or limitation of how long an implied warranty lasts, so the above exclusion and limitation may not apply to you.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2001 Invacare Corporation
Invacare, SensO₂, and Venture are registered trademarks of Invacare Corporation.
Homefill, Sieve-Gard, and Yes, you can. are trademarks of Invacare Corporation.